



SEMI-PERMEABLE TRANSPARENT DRESSING

Indication: Covering superficial wounds and surgical wounds


- Excellent adhesiveness
- Soft and ventilated backing
- Waterproof and bacteriaproof
- Painless removal

Optiskin®

Supplied in boxes of individually pouched and sterile dressings, ready to use

SIZES	PAD SIZES	UNITS PER BOX
5.3 x 7 cm	2.9 x 3.5 cm	50 pouches
10 x 7 cm	7 x 3.5 cm	50 pouches
10 x 9 cm	7 x 4.5 cm	50 pouches
15 x 9 cm	11.5 x 4.5 cm	20 pouches
20 x 9 cm	15.8 x 4.5 cm	20 pouches
25 x 9 cm	19.4 x 4.5 cm	20 pouches
30 x 9 cm	23 x 4.5 cm	20 pouches

Optiskin®

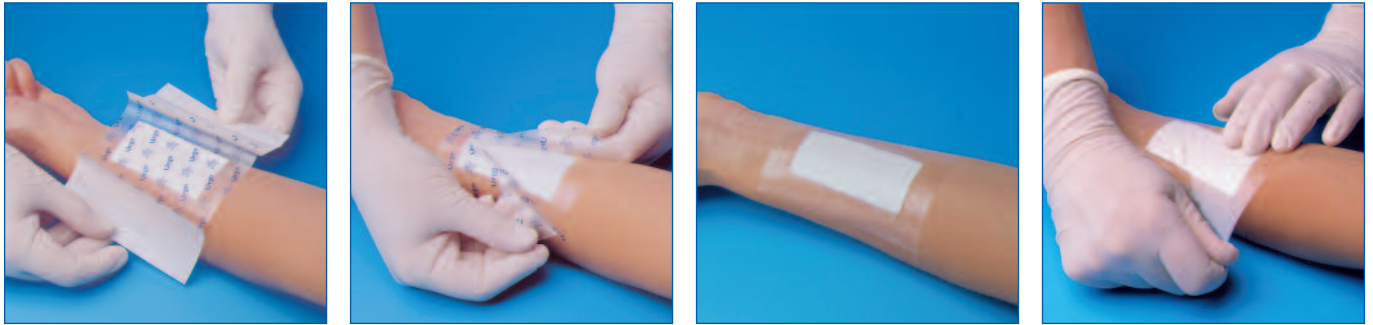

DESCRIPTION

- Sterile semi-permeable transparent dressing.

COMPOSITION: Optiskin® is a dressing made of a thin polyurethane film coated with an acrylic adhesive mass, and a low adherent non-woven pad. The dressing is covered with a transparent polypropylene film. The adhesive mass is protected by two release papers.

METHOD OF STERILIZATION: Sterilized by ethylene oxide.

PROPERTIES

- Optiskin® is waterproof, allowing the patient to shower.
- Bacteriaproof, Optiskin® eliminates the risks of external contaminants.
- Semi-permeable, it allows gas exchange and water vapour, preventing from maceration.
- Flexible and conformable, it fits in all areas of the body.
- Hypoallergenic, it is gentle on sensitive skin.
- Transparent, it allows monitoring of the saturation level of the pad.
- Its long lasting adhesiveness enables Optiskin® to stay in place up to 5 days.
- Its polypropylene film facilitates application of the dressing.
- Painless on removal, it enables repetitive applications, while still respecting the skin.

INDICATIONS

- Optiskin® is indicated for covering any type of superficial wounds (sutured wounds, skin abrasions) and any type of device (probes, drains...).
- Waterproof, Optiskin® is particularly suitable for balneotherapy, or for quick resumption of activity after surgery.

DIRECTIONS FOR USE

METHOD OF USE

- Cleanse the wound with saline solution.
- If an antiseptic is first used, rinse the wound thoroughly with saline solution before applying Optiskin®. Dry thoroughly.
- If necessary, shave the skin to ensure good adhesion of the dressing.
- If necessary, proceed to haemostasis of the wound.
- Holding the protective papers between the thumb and the forefinger, carefully expose the adhesive side of the dressing by pulling back each side.
- Position the dressing and apply it onto the wound/device, before having fully removed the protective papers. Avoid tensions when applying Optiskin®.
- Apply a gentle pressure over the whole surface to ensure adhesiveness.
- Remove the transparent film, starting at the central lift.
- Smooth down the dressing from the centre towards the edges.
- To remove, lift one edge and gently pull it to stretch and release adhesion. This technique will provide a painless removal, avoiding skin irritation.
- Optiskin® can be left in place up to 5 days.

PRECAUTIONS

- Do not use on infected wounds, sites, bleeding wounds or highly exuding wounds.
- Check the wound and surrounding skin regularly to detect any local signs of infection.
- Optiskin® should be stored away from light, heat and moisture.
- Do not use if sterile protection has been damaged.

LEGAL STATUS

CE 0459 Medical device