

Urgotul®



LIPIDO-COLLOID CONTACT LAYER

Indication: Low to moderate exuding wounds


- Healing in a moist environment
- Painless and atraumatic removal
 - Proliferation of fibroblasts

Urgotul®

Supplied in boxes of individually pouched and sterile dressings, ready to use

SIZES	UNITS PER BOX
5 x 5 cm	10 pouches
10 x 10 cm	10 pouches
15 x 20 cm	10 pouches
10 x 40 cm	10 pouches

Urgotul®

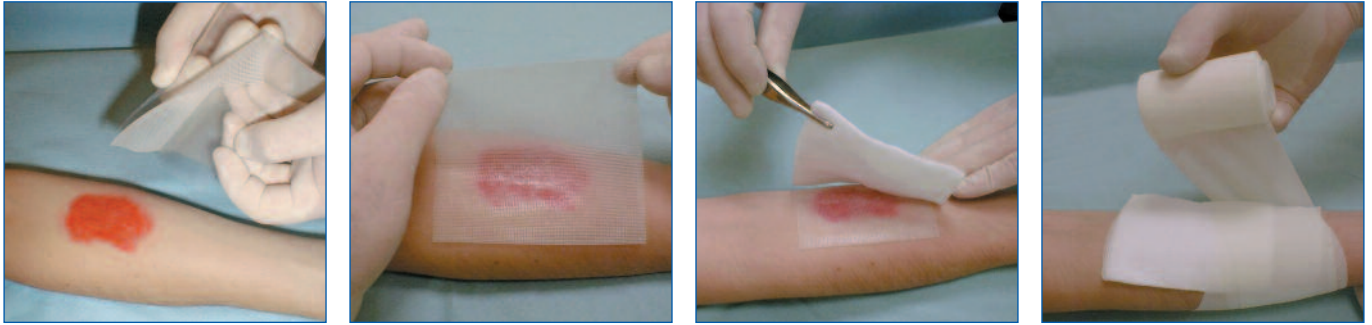

DESCRIPTION

- Sterile non-occlusive, non-adhesive, lipido-colloid contact layer (issued from the Lipido-Colloid Technology, exclusive patented innovation by Urgo Laboratories).

COMPOSITION: Urgotul® is a lipido-colloid dressing made of a polyester mesh impregnated with hydrocolloid (carboxymethylcellulose) and vaseline particles.

METHOD OF STERILIZATION: Sterilized by ionizing radiation.

PROPERTIES

- In contact with wound exudate, the hydrocolloid particles from **Urgotul®** interact with the petroleum jelly component, to form a lipido-colloid gel creating a moist environment favourable for the healing process.
- Fatty in its chemical composition, without being greasy to the touch, **Urgotul®** does not adhere to the wound or to the surrounding skin. Removal of the dressing is painless and atraumatic for the wound.
- **Urgotul®** promotes fast healing by stimulating fibroblasts proliferation.
- Flexible and conformable, **Urgotul®** is especially suitable for wounds located in awkward areas.

INDICATIONS

- **Urgotul®** is indicated for the local treatment of acute wounds (burns, skin abrasions, traumatic wounds...) and chronic wounds (leg ulcers, pressure ulcers...) at the granulation/epithelialisation stage.
- **Urgotul®** is also indicated for the treatment of wounds caused by congenital epidermolysis bullosa.

DIRECTIONS FOR USE

METHOD OF USE

- Cleanse the wound with saline solution.
- If an antiseptic is first used, rinse the wound thoroughly with saline solution before applying **Urgotul®**.
- Remove the protective transparent films from the dressing.
- Apply **Urgotul®** directly onto the wound. If needed, **Urgotul®** can be cut using sterile scissors to match the size of the dressing to the wound.
- Cover **Urgotul®** with a secondary dressing (sterile pad suitable for absorption of exudate).
- Secure the secondary dressing with a retention bandage (Nylex®).
- **Urgotul®** should be changed every 2 to 4 days, up to 7 days depending on the condition of the wound. In case of epidermolysis bullosa, **Urgotul®** should be changed every 1 to 3 days.

PRECAUTIONS

- **Urgotul®** may adhere to latex gloves. In that case, moistening the gloves with saline solution will facilitate handling of the dressing.
- If clinical signs of local infection appear, the treatment may be continued under medical supervision.
- However, it is preferable to use **Urgotul® SSD/S.Ag** or **Urgotul® Silver** instead.
- In case of deep wounds, **Urgotul®** does not need to be applied in several layers.
- **Urgotul®** should be stored flat, away from light, heat and moisture.

LEGAL STATUS

CE 0459 Medical device