

# Urgotul<sup>®</sup> Duo Silver



## LIPIDO-COLLOID ANTIBACTERIAL CONTACT LAYER WITH A PROTECTIVE PAD

Indication: Low to moderate exuding wounds at risk of infection

- Antibacterial efficacy on a large spectrum of bacteria
  - Healing in a moist environment
  - Painless and atraumatic removal
  - Protection of the wound

## Urgotul<sup>®</sup> Duo Silver

Supplied in boxes of individually pouched and sterile dressings, ready to use

SIZES	UNITS PER BOX
5 x 7 cm	26 pouches
11 x 11 cm	16 pouches
15 x 20 cm	10 pouches

# Urgotul® Duo Silver



## DESCRIPTION

■ Absorbent sterile non-occlusive, non-adhesive, antibacterial lipido-colloid dressing (issued from the Lipido-Colloid Technology, exclusive patented innovation by Urgo Laboratories).

**COMPOSITION:** Urgotul® Duo Silver is a lipido-colloid dressing made of a polyester mesh impregnated with hydrocolloid (carboxymethylcellulose), vaseline and silver salts, combined with a protective absorbent pad (100% viscose).

**METHOD OF STERILIZATION:** Sterilized by ionizing radiation.

## PROPERTIES

- The silver salts provide a bactericidal effect on most of the microorganisms involved in secondary infection: Gram-positive (MRSA, Staphylococcus Aureus...), Gram-negative (Pseudomonas Aeruginosa...), and yeasts.
- In contact with wound exudate, the hydrocolloid particles from **Urgotul® Duo Silver** contact layer interact with the petroleum jelly component, to form a lipido-colloid gel creating a moist environment favourable for the healing process. The silver ions are maintained in this lipido-colloid gel, providing an antibacterial activity on contact with the wound only.
- Fatty in its chemical composition, without being greasy to the touch, **Urgotul® Duo Silver** does not adhere to the wound or to the surrounding skin. Removal of the dressing is painless and atraumatic for the wound.
- The pad of **Urgotul® Duo Silver** protects the wound and absorbs exudate from the wound. Application is easier, as there is no need to apply a secondary dressing. It is also more comfortable for the patient.
- The transparent films enable to apply **Urgotul® Duo Silver** without touching the mesh and without sticking to latex gloves.
- Flexible and conformable, **Urgotul® Duo Silver** is especially suitable for wounds located in awkward areas.

## INDICATIONS

- **Urgotul® Duo Silver** is indicated for the local treatment of low exuding acute wounds (burns, skin abrasions, traumatic wounds...) and chronic wounds (leg ulcers, pressure ulcers...) at the granulation/epithelialisation stage at risk of infection.

## DIRECTIONS FOR USE

### METHOD OF USE

- Cleanse the wound with saline solution.
- If an antiseptic is first used, rinse the wound thoroughly with saline solution before applying **Urgotul® Duo Silver**.
- Remove the protective transparent films from the dressing.
- Apply **Urgotul® Duo Silver** directly onto the wound. If needed, **Urgotul® Duo Silver** can be cut using sterile scissors to match to the size of the wound.
- Secure **Urgotul® Duo Silver** with a retention bandage (Nylex®) or adhesive plaster.
- **Urgotul® Duo Silver** should be changed every 1 to 3 days, depending on the condition of the wound.

### PRECAUTIONS

- Treatment with **Urgotul® Duo Silver** should be carried out under medical supervision.
- Concomitant use with other local treatments is not recommended.
- Use on pregnant women, breast-feeding, newborn and neonates is contraindicated in the absence of specific clinical information.
- Avoid contact with electrodes or conducting gels, during electronic measurements (EEG, ECG).
- Do not use on patients undergoing Magnetic Resonance Imaging (MRI) examination.
- **Urgotul® Duo Silver** should be stored flat, away from light, heat and moisture.
- Do not fold **Urgotul® Duo Silver**.
- Do not use if sterile protection has been damaged.

## LEGAL STATUS

CE 0086 Medical device